



NuMask[®]

It's a matter of life and breath.


CPR Instructions

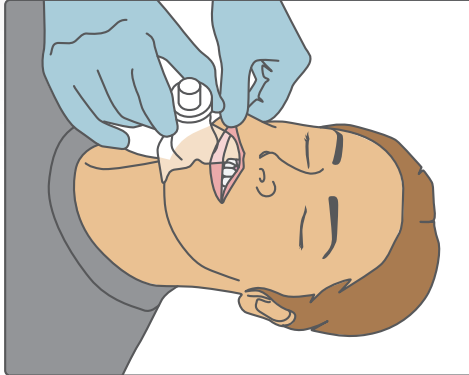


STEP 1: Elevate lateral edge of the lips and slide in a flap of the IOM®.

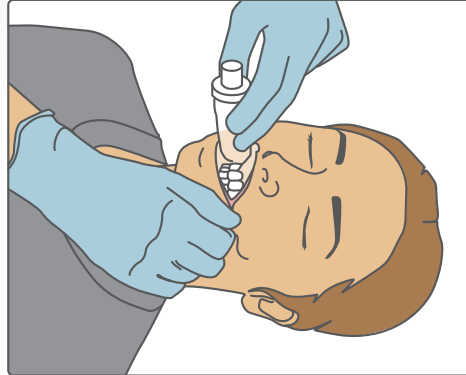

STEP 2: Grasp and elevate the other side of lips to insert remainder of IOM.

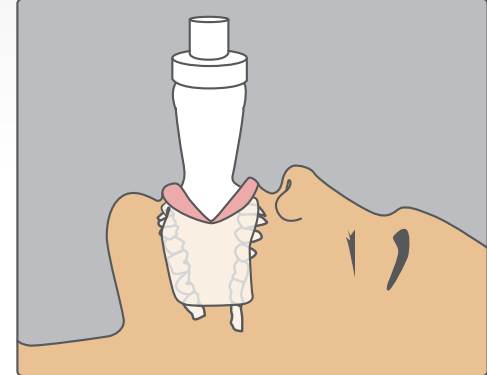


STEP 3: IOM sits in front of teeth & behind lips.

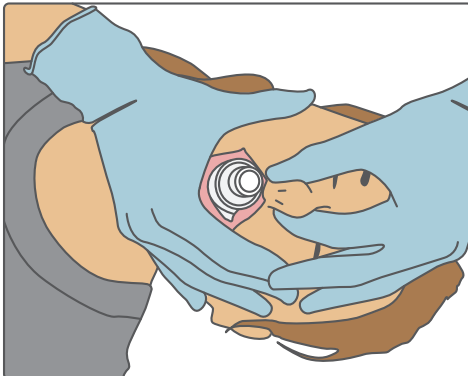

STEP 4: Fingers and hands surround and close the lips around the stem of the mask. Pinch the nose closed with the other hand.



STEP 5: If not contraindicated, place victim in head tilt/chin lift position, then deliver rescue breaths.

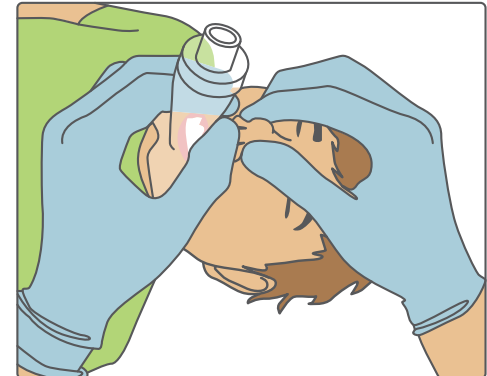

PEDIATRICS: For infants and children, the mask is placed and sealed on top of the open lips. The nose is pinched closed with the other hand.