



NuMask[®]

It's a matter of life and breath.

RETENTION SHIELD Instructions

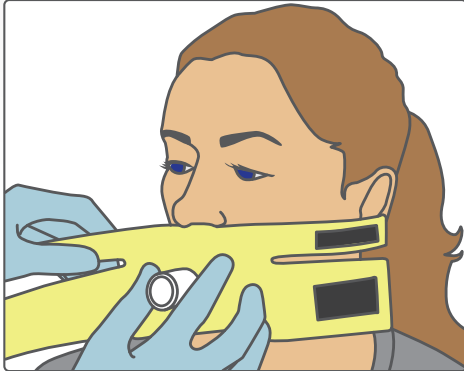


STEP 1: First insert IOM[®]. Slide the shield over the IOM so that the shield notch goes up into the nose and the raised surface of the shield hole contacts the lips. Velcro faces away from patient.

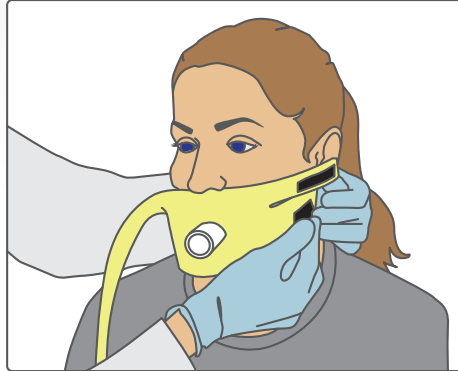


STEP 2: Fasten the lower velcro strap around the back of the neck.




STEP 3: Fasten the upper velcro strap around the back of the head so that the strap sits above the ear and the shield occludes the nose.

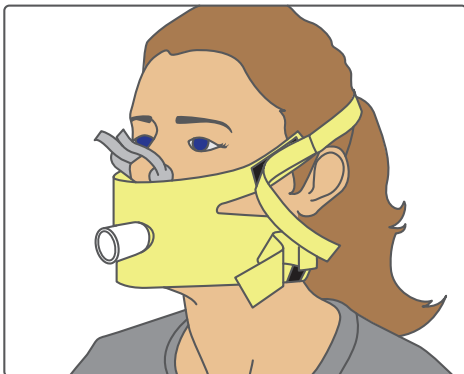

STEP 4: Attach nose clip if shield does not fully occlude nose.

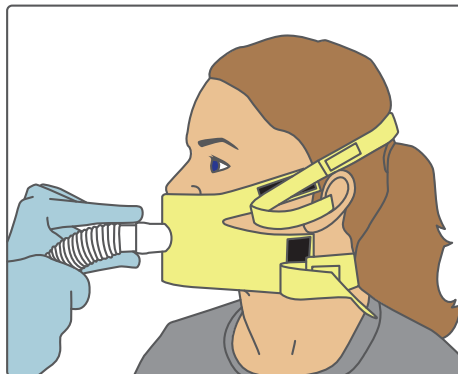

STEP 5a: Attach standard CPAP/BiPAP or anesthesia circuitry directly to the IOM.

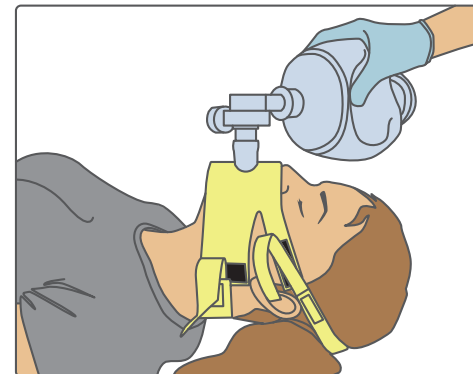

STEP 5b: Attach a resuscitator bag for hands free ventilation during transport, rescue and evacuation.